





SOLID VALVE

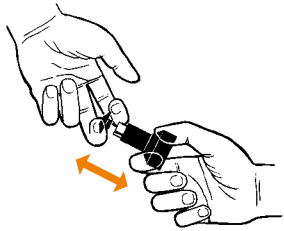


1. Pull off Lip Piece



2. Dismantled Mouthpiece

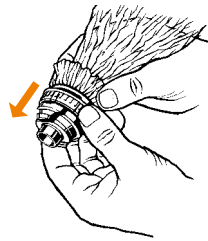


3. Pull off Slirping and Balloon

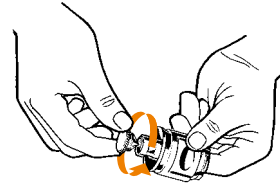


4. Screw off Valve Cap




5. Dismantled Valve

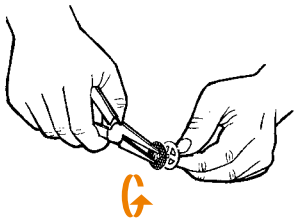


6. Screw off nut from Filling Chamber Insert

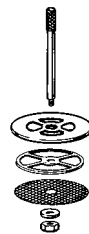

7. Dismantled Filling Chamber Insert

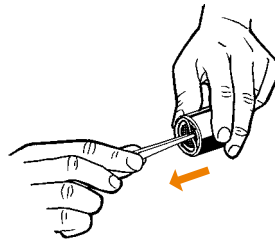

8. Pull off Filling Chamber Ring

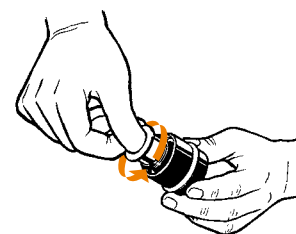



9. Screw off Filling Chamber Cylinder




10. Dismantled Filling Chamber




11. Manual cleaning with alcohol (min. 90%), also dishwasher safe

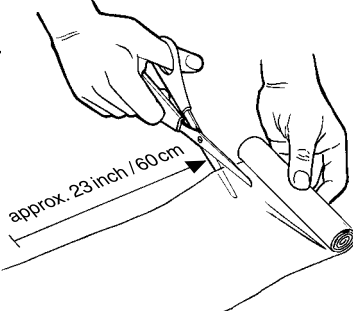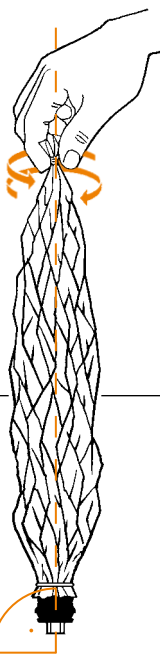

12. Cut off a piece of approx. 50-60 cm (19-23 inch)

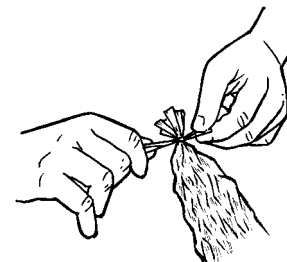

13. Tie close at one end

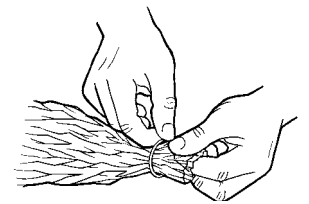


14. Attach Balloon Fixation Ring




15. Attach Balloon to Valve


16. Align Valve Balloon to Valve

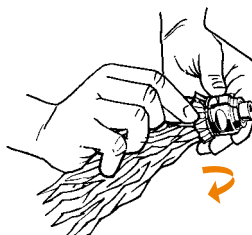

17. Evert balloon mantle

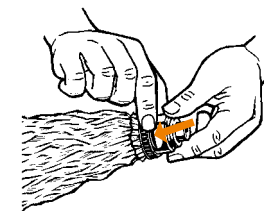

18. Attach Slirping to secure Balloon

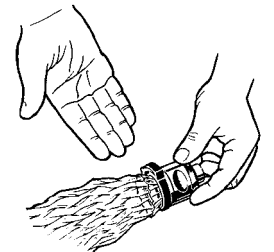

19. Ready to use